



Permitted Cleaning Agents for GE Healthcare Clinical Accessories

Used with Patient Monitoring and Anesthesia and Respiratory equipment

As preferred cleaning practices change and new cleaning products become available, GE Healthcare continues to test the compatibility of cleaning agents with GE Healthcare Clinical accessories. The tables below provide an overview of the various cleaning agents we have tested and the determination of those tests for GE Healthcare clinical accessories.

The permitted agents are also available on [GE Healthcare cleaning web site](#).

For cleaning procedures, please refer to your hospital procedures. You can find additional recommendations in the Instruction for Use (IFU), available on the [customer site](#).



Accessories for Anesthesia and Respiratory

Legend : "A" = Permitted " " = Not tested "N" = Not permitted	Sub-Group / Details	P/N's	Sterilisation - Autoclave *	Specific recommendations (see Instruction for Use)	Concentrated acids and alkalines	Phenols and Phenols derivatives	Hydrogene Peroxyde and Ozone	Orgnic Solvents (eg alcohols, ketones, ether, etc)	Aldehydes	Active Chlorine	Active Oxygen	Peracetic acid	Desinfectant and Biocides	detergent solutions or a mild alkaline or pH neutral cleaner
	Patient circuits	Patient circuits Kits	2096534-009 , 2096536-001	A										
	Patient circuits	Patient circuits Kits	2096534-101,-102,-103	A										
	Patient circuits	Patient circuits Kits	2096534-104	A										
	Patient circuits	Patient circuits Kits	2096536-002	A										
	Patient circuits	Patient circuits Accessories	2105488-002, -003 ,004		A									
	Bag	Reusable Bag Arm Connector	2105488-005	A		N	N	N	N	A	A	A	A	
	Bag	Manual bag	2106570-001,-002,-003, -004, 025,-026,-027,-028	A		N	N	N	N	A	A	A	A	
	Mask	Mask reusable	2106570-011,-019,-020, -021,-022,-023, -029,-030, -031,-032,-033,-034	A									A	
	Bag	Scavenging bag	2106570-024	A										A

* : Always remove organic material and residual or disinfecting chemical before autoclaving